

Adult Sizing Chart

IMPORTANT PRINTING INSTRUCTIONS

Make sure to set page scaling to **None**, or turn off **Fit To Page** option.

To check that your sizing chart printed accurately, the printed ruler should measure **8"** or **20.3cm** long.

HOW TO FIND SHOE SIZE

Place heel on chart

- Align heel of foot/AFO/KAFO with the heel line.
- If foot is longer than brace, include foot

Trace foot/AFO/KAFO

- Match longest point to toe line. Solid lines are full sizes, dashed lines are half sizes.
- If you are between or slightly above line, round up

Size Adjustments

- **Women Size: Add 1 full size**
- Articulating hinges, posterior stop and instep strapping: Add 1 full size
- Toe strapping, the additions above will change the fit of the shoe: Add 1/2 full sizes



APIS Footwear Co.

2239 Tyler Ave., South El Monte, CA 91733

Phone: (888) 937-2747 Fax: (888) 990-2245

sales@apisfootwear.com, orders@apisfootwear.com

<https://apisfootwear.com>

